



Rating-Box – Emotion Sensor for Biofeedback

*Connect Psychological Questions
with Biofeedback*



... *Render Emotional States Visible* ...

Included in Delivery

- Rating-Box
- Connection cable for the ProComp™ Encoder, e.g. ProComp™ 2, ProComp™ 5 or ProComp Inf niti™
- Mini-Suite for BioGraph Inf niti™

Optional Accessories:

Push Button: With the Push Button the practitioner can label a concrete result.

Note:

In addition to the Rating Box, an Encoder and the Bio-Graph Inf niti™ Software are required for use.

- I would like to order the Rating Box (incl. Software) at the price of 299.00 Euros (plus VAT and shipment)
- I would like a no-obligation presentation on bio- and neuro-feedback equipment – Please contact me to make an appointment.
- I would like more specific information about bio- and neuro-feedback. Please send me additional information.

Last Name:

First Name:

Date of Birth:

Street:

Zip Code / City:

Telephone:

Email:

Date, Signature

Please send your order to:

MediTECH Electronic GmbH
Langer Acker 7 Tel.: 05130 – 9 77 78-0
30900 Wedemark Fax: 05130 – 9 77 78-22
service@meditech.de www.meditech.de
Or place your order conveniently online at
www.shop.meditech.de

Recommended by:



MediTECH Electronic GmbH
Langer Acker 7
30900 Wedemark (OT Bissendorf)

Telefon: +49 (0)5130-97778-0
Fax: +49 (0)5130-97778-22
E-Mail: service@meditech.de
Internet: www.meditech.de

The Approach

How are your clients during biofeedback sessions?
Are you still able after two weeks to reconstruct how anxious or relaxed he or she felt in a specific situation – or do you depend on deduction?

Using the Rating Box, the client can give a non-verbal response to a question or subjectively evaluate a situation by engaging a control dial during the session. The ProComp™ device records and transforms input into curves and/or numerical values that, like other sensor data, are displayed on the monitor and can be processed. The response to an exercise or answer to a question posed by the practitioner thus becomes visible, measurable and able to be documented.



The Unit

The Rating Box is a small, easy to use box that is connected to the Biofeedback equipment with a sensor cable. The clients turn the control dial from the neutral position to the left or right, depending on the answer to the question. In order to ensure the most differentiated/ refined answers possible can be given, setting the dial is smooth and continuous. The connection to the biofeedback system is very easy: the sensor cable is plugged into one of the input ports on the ProComp™ Encoder. The Rating Box is recognized at each sensory input port as the sensor defined for that particular channel and can therefore be individually used as needed.



Using the Rating Box

There is no limit to the variety of questions that can be individually customized to the client and the situation. The range of answers can also be widely diversified:

Possible Questions	Answer Range From ... To
How do you feel right now?	Quite well ... Quite unwell
What mood are you in?	Very happy ... Very depressed
How would you rate the following statement?	Strongly agree ... Strongly disagree
How would you rate the scene?	As very good ... As very bad
Is that correct?	Yes, Maybe, No

In addition, questions that lend themselves to subjective self-assessment and that can be examined and compared by objective measurements can also be used. Statements regarding situation-specific physical awareness can also be easily compared.

How brave / afraid do you currently feel?	Very brave ... Extremely terrified
How would you rate your level of stress right now?	Very relaxed ... Very tense

